Nathanson Liver Retractors

Instructions for Use

EN
3

Nathanson leverretraktorer

Brugsanvisning

DA
3

Nathanson Leberretraktoren

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DE
4

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C _ T _ N L R S _ R E V . 2

NATHANSON LIVER RETRACTORS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Nathanson Liver Retractors are constructed of stainless steel and are anodized to eliminate light reflection. The product is available in two sizes.

INTENDED USE

The Nathanson Liver Retractors are intended to support and retract the liver during laparoscopic upper GI surgery. The product is intended for use by physicians trained and experienced in laparoscopic surgical techniques. Standard techniques should be employed.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

None known

PRODUCT RECOMMENDATIONS

Use with a Retractor Arm

The Nathanson Liver Retractors can be held in place by any of the following commercially available mechanical arms:

- The Murdoch Arm—Dedicated Locking Handle
- The Bookwalter® Retractor—Clamp Code 50-4582, which attaches to the horizontal flex bar (**NOTE:** Bookwalter is a registered trademark of Codman and Shurtleff, Inc.)
- The Iron Intern®—The Universal Gemma Connection Clamp Code NSN6515-01, 100-5561 G9 (**NOTE:** Iron Intern is a registered trademark of Automated Medical Products Corp.)

NOTE: The retractor arm should be placed on the opposite side of the table from the surgeon's working side.

Sterilization Parameters

The Nathanson Liver Retractors can be sterilized by steam autoclave at 121 degrees C for 15 minutes. The retractors should be thoroughly cleaned and decontaminated prior to sterilization.

INSTRUCTIONS FOR USE

1. Insufflate the peritoneal cavity in the normal manner.
2. Upon obtaining set maximum pressure, insert a 5 mm trocar at the appropriate location.
NOTE: The insertion point for the liver retractor depends upon the procedure being performed, size of the patient and personal preference of the surgeon. For example, when performing a Nissen fundoplication, an optimal site may be at the point inferior to the xiphoid process.
3. Remove the trocar and enlarge the tract, if necessary, with a pair of artery forceps.
4. Holding the liver retractor by the handle and maintaining the curved radius in a vertical position, insert the retractor with a twisting motion.
5. Under direct laparoscopic vision, maneuver the curved edge of the liver retractor in a superior direction under the left lobe of the liver. **Do not connect the liver retractor to the retractor arm at this point.**
6. Continue placement of the ports. When this is complete, attach the liver retractor to the retractor arm and elevate the liver to the desired position. Periodic adjustment of the retractor tension may be necessary throughout the procedure.

HOW SUPPLIED

Supplied non-sterile. Reusable. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.