COOK
MEDICAL

CE
0088

EN
3

One-Step Suprapubic Introducer

Instructions for Use

CS
4

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DA
6

Ettrins suprapubisk indfører

Brugsanvisning

DE
7

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T _ O S I _ R E V 1

ONE-STEP SUPRAPUBIC INTRODUCER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Trocar Stylet
- Dilator
- Peel-Away® Sheath

NOTE: Set components may vary.

INTENDED USE

Used for introducing a drainage catheter suprapubically into the bladder.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Introducers are intended for use by physicians trained and experienced in techniques for suprapubic access.
- Ensure patient has a full and palpable bladder for correct bladder position.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

POTENTIAL ADVERSE EVENTS

Complications that might occur during this procedure could include incorrect location at insertion, resulting in trauma to the surrounding tissue.

INSTRUCTIONS FOR USE

1. Using aseptic technique, remove the introducer from its outer packaging and place in the sterile field. Prepare the puncture site in the usual manner. Ensure the patient has a full and palpable bladder to confirm urine return.
2. Inspect the introducer to ensure no damage has occurred during shipment. Defective devices should be returned to Cook.
NOTE: Remove the needle tip protector prior to insertion.
3. Ensure components are assembled into one piece.
4. Use a scalpel to make a small incision at the puncture site. Advance the assembled components with one single pass suprapubically into the bladder.
5. Remove the trocar stylet to confirm the device is within the bladder by return of urine flow. If spontaneous urine flow is not established, a syringe may be attached to the dilator for aspiration.
NOTE: Only proceed if a flow of urine is confirmed.

6. Remove the dilator from the Peel-Away Sheath to allow placement of an appropriately sized drainage catheter through the sheath.

CAUTION: The outer diameter of drainage catheters vary from manufacturer to manufacturer. Care should be taken to verify the drainage catheter will pass through the Peel-Away Sheath prior to suprapubic insertion.

7. Insert the drainage catheter as usual through the sheath. Using each side of the Peel-Away Sheath, gently pull back and split the sheath into two parts while leaving the catheter in situ.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.