Quick-Core® Biopsy Needle

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Quick-Core® biopsinål

Quick-Core® Biopsiekanüle

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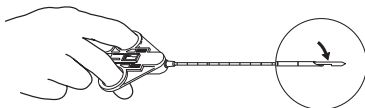
Figure D

QUICK-CORE® BIOPSY NEEDLE

Used to obtain core biopsies of prostatic tissue. Lightweight and compact, Quick-Core® Biopsy Needle design facilitates one-handed operation. Precise, rapid firing mechanism provides intact core tissue sample. Beveled point stylet permits easy penetration into the specimen with less trauma to surrounding tissue. Etched tip enhances visualization during ultrasound imaging. Intended for single procedure use.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING QUICK-CORE® BIOPSY NEEDLE

1. Prior to insertion, prepare the Quick-Core® Biopsy Needle by pulling back on the plunger until firm click is felt indicating that the needle spring is locked into ready position. **(Figure A)**
2. With the stylet fully retracted, so that the specimen notch is completely covered by the cutting cannula, advance the needle proximal to the area to be biopsied. **Do not advance stylet until Quick-Core® Biopsy Needle is in position. (Figure B)**
3. While maintaining needle position, advance stylet with thumb, to expose specimen notch within the area to be biopsied. Fire the cutting cannula by fully depressing the plunger with thumb to capture tissue within the specimen notch. **(Figure C)**
4. Withdraw needle from biopsy area. To remove tissue specimen, pull back on the plunger until firm click is felt to indicate the cutting cannula is locked into position. Push stylet forward to expose tissue specimen within the notch. Remove tissue from specimen notch. **(Figure D)**

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.