Cook® Continance Cuff

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Cook® kontinensmanchet

Cook® Kontinenzmanschette

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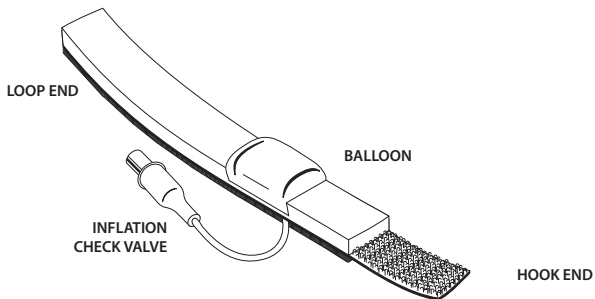
T _ C C C _ R E V O

COOK® CONTINENCE CUFF

Used for control of male urinary incontinence due to surgery or neurogenic disorder. Reusable. Intended for single patient use only.

CONTENTS

- Continence Cuff
- Syringe

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING COOK® CONTINENCE CUFF

NOTE: The patient should use this product only after receiving complete instructions from a physician and has a complete understanding of product and instructions.

1. The Continence Cuff should be placed 1 to 2 fingers from the scrotum.
2. Wrap the cuff around the penis. Determine the proper cuff length and adjust by cutting off excess material at the loop end. **(Figure A)**

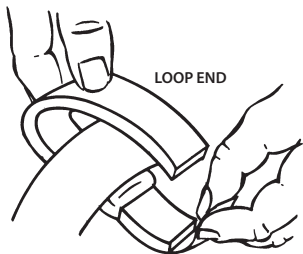
Figure A

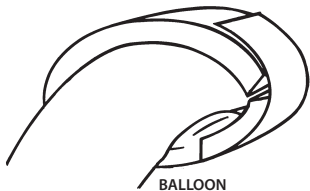
Figure B

3. Wrap the adjusted cuff around the penis. Ensure that the balloon is below the urethra. (Figure B)
4. Inflate the balloon with the syringe until continence is achieved.
5. Record the volume of air needed to properly inflate the balloon. Use the same amount of air each time the cuff is placed. When removing the cuff, withdraw the exact amount of air used for inflation. The air left in the balloon will affect the next inflation.

CAUTION: Overinflation of the balloon can impair blood circulation which may not be immediately observable. If any unusual skin irritation, discoloration, or pain occurs, contact your physician immediately.

6. Deflate or remove the cuff at least every 3 to 4 hours.

NOTE: If gradual balloon deflation is noticed, it may be necessary to flush the inflation check valve with 3 to 4 ml of water to remove debris. Inject water into inflation check valve (with syringe) and then withdraw water. Repeat this process 2 to 3 times. Remove water before reusing. Reinflate with the proper volume of air as previously noted.

NOTE: This device is not recommended for use during sleep.

CLEANING INSTRUCTIONS

1. Hand wash in mild soap and water. (Do not machine wash.)
2. Rinse well. (Rinse until all soap residue has been removed.)
3. Air dry. (Do not machine dry.)

CAUTIONS

This device is not intended for use with other continence devices or indwelling catheters.

This device is intended for control of urinary incontinence only.

NOTE: Patient direct orders can be obtained through Cook® with written approval from a physician. If you have any questions, please contact your physician or Cook®.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.