UMI – Uterine Manipulator Injector
 UMI-Loc™ – Lockable Uterine Manipulator Injector
 UMI – Děložní manipulátor a injektor
 UMI-Loc™ – Zamykatelný děložní manipulátor a injektor
 UMI – Uterin manipulator/injektor
 UMI-Loc™ – Låsbar uterin manipulator/injektor
 UMI – Uterusmanipulator/Injektor
 UMI-Loc™ – Feststellbarer Uterusmanipulator/Injektor
 UMI – Εργαλείο χειρισμού/εγχυτήρας μήτρας
 UMI-Loc™ – Ασφαλιζόμενο εργαλείο χειρισμού/εγχυτήρας μήτρας
 UMI – Manipulador-inyector uterino
 UMI-Loc™ – Manipulador-inyector uterino bloqueable
 UMI – Injecteur manipulateur utérin
 UMI-Loc™ – Injecteur manipulateur utérin verrouillable
 UMI – Méhmanipulátor/injektor
 UMI-Loc™ – Zárható méhmanipulátor/injektor
 Manipolatore/iniettore uterino UMI
 Manipolatore/iniettore uterino bloccabile UMI-Loc™
 UMI – Uteriene manipulator/injektor
 UMI-Loc™ – Vergrendelbare uteriene manipulator/injektor
 UMI – uterin manipulator/injektor
 UMI-Loc™ – låsbar uterin manipulator/injektor
 UMI – Manipulator/wstrzykiwacz maciczny
 UMI-Loc™ – Manipulator/wstrzykiwacz maciczny z blokadą
 UMI — Manipulador/injektor uterino
 UMI-Loc™ — Manipulador/injektor uterino bloqueável
 UMI – uterin manipulerare/injektor
 UMI-Loc™ – Låsbar uterin manipulerare/injektor
 UMI – 子宫操作器/注射器
 UMI-Loc™ – 可锁定子宫操作器/注射器

EN
3
 CS
4
 DA
5
 DE
7
 EL
9
 ES
10
 FR
12
 HU
14
 IT
15
 NL
17
 NO
18
 PL
20
 PT
21
 SV
23
 ZH
24

T _ U M I _ R E V O

UMI – Uterine Manipulator Injector

UMI-Loc™ – Lockable Uterine Manipulator Injector

Used for manipulation or repositioning of the uterus during diagnostic and surgical procedures, and for the injection of medium into the uterine for diagnostic purposes. Intended for one-time use.

DESCRIPTION

UMI – 4.5mm curved catheter 33cm long with a 10mL distal balloon and a removable, repositionable rigid handle with a flat cervical stop. Tip is open to permit uterine instillation of medium.

UMI-Loc – 5.0mm curved catheter 37cm long with a 10mL distal balloon and a sliding handle that may be locked to the catheter shaft. The sliding handle has a flat cervical stop. Tip is open to permit uterine instillation of medium.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is damaged.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

INDICATIONS

Indicated for manipulation of the uterus during laparoscopy, minilaparotomy, fertility examination, and salpingoplastic procedures. Indicated for sealing of the internal cervical os to inject medium into the uterus and fallopian tubes.

CONTRAINDICATIONS

This device should not be used in the presence of profuse bleeding, pregnancy, active pelvic infection, sexually transmitted disease, or other conditions where cervical passage is contraindicated.

WARNING: Excessive force in manipulation may result in uterine perforation or may cause the product to bend or break, resulting in uterine or vaginal trauma.

WARNING: Neglecting to securely attach or lock the handle at the correct sounded depth may result in uterine trauma during manipulation.

WARNING: Inflate the distal balloon prior to manipulation of the uterus to avoid trauma.

WARNING: Insert the catheter along the proper axis to avoid uterine trauma.

ADVERSE REACTIONS

- Cramping
- Infection
- Uterine trauma or perforation
- Uterine spasm
- Vasovagal syncope

SUGGESTED INSTRUCTIONS FOR USING UMI and/or UMI-Loc™ UTERINE MANIPULATOR INJECTORS

NOTE: UMI-Loc – Ensure the sliding handle is not locked and slides smoothly.

NOTE: Test distal balloon by injecting up to 10mL of sterile saline in the inflation valve. Deflate balloon after test.

1. Sound the uterus for depth and direction.
2. Dilate cervix to appropriate size, if necessary.
3. Lubricate catheter tip and balloon to ease insertion and cervical passage.
4. Position proximal end of the sliding handle on the catheter shaft at the same centimeter depth markings as sounded uterine depth.

5. Guide the uterine manipulator injector along the natural axis of the cervix carefully to avoid cervical or uterine injury or perforation.
6. While maintaining the sounded depth handle position, advance the catheter into the uterus until the cervical stop presses on the cervix. Slowly inflate the distal balloon with a syringe by injecting 3mL-10mL of sterile saline through the inflation valve.
7. UMI-Loc™ – Slowly advance the sliding handle while holding the catheter shaft to push the cervical stop against the cervix. Lock the sliding handle to the shaft by snapping the locking tab onto its mating part.
NOTE: Depth markings on catheter shaft are approximate. Sound surgical technique should include care upon passage to avoid uterine perforation.
8. To insure that the distal balloon has not ruptured during cervical passage or the procedure, check the inflation valve pilot balloon. A soft pilot balloon indicates a ruptured or leaking distal balloon.
NOTE: A deflated distal balloon increases the danger of uterine trauma.
9. Grasp the handle to manipulate the uterus.
10. Contrast or distension medium may be injected through the Luer Lock adapter at the proximal end of the catheter shaft.
11. To remove the uterine manipulator injector, deflate distal balloon through the inflation valve. Carefully withdraw the instrument.
12. Check before discarding the catheter to be sure it is intact and no parts remain in the patient.

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Catheter Research, Inc.
D.B.A. Thomas Medical, Inc.